# FUSION<sup>2K</sup> / <sup>2K</sup> XL

## Operations Manual

- FUS2K80
- FUS2K80CA
- FUS2K84
- FUS2K84CA
- FUS2KXL4280
- FUS2KXL4280CA
- FUS2KXL4880
- FUS2KXL4880CA

### 1 Year Product Warranty

MOXI Enterprises, LLC warrants the FUSION 2K and 2K XL Series for a period of 1 year (12 months) from date of purchase.

The mattress component and control unit is warranted for a limited twelve (12) month period. Normal usage is not covered under warranty. Mattresses subjected to harsh cleaning chemicals, such as bleach, are not covered under warranty. The mattress cover, and all other mattress system components will be warranted for a period of twelve (12) months.

This warranty does not apply to any products that have been subjected to damage in accident, misuse or abuse.

For additional information, please contact MOXI at (800) 942-7678, (314) 771-5222 or by fax at (314) 771-5225.

Manufactured by **MOXI**™ in the USA   
**ENTERPRISES**  
 moxiusa.com

**MOXI Enterprises** | 1939 S. Vandeventer Ave., St. Louis, MO 63110  
 moxiusa.com | 1-800-942-7678 | p.314.771.5222 | f.314.771.5225


### Mattress Setup:

- (1) System comes complete.
- (2) Place directly on an appropriate size bed frame.
- (3) Connect hose firmly into hose port.
- (4) Please see *SelectAir*® manual for complete *SelectAir*® pump operation.

### Cleaning Protocol:

- (1) Cover may be wiped down with a mild anti-bacterial cleaning solution.
- (2) All other components may be wiped down with a mild anti-bacterial cleaning solution.

These instructions are intended only as general guidelines. Follow your facilities standard protocol for cleaning and disinfecting medical equipment.

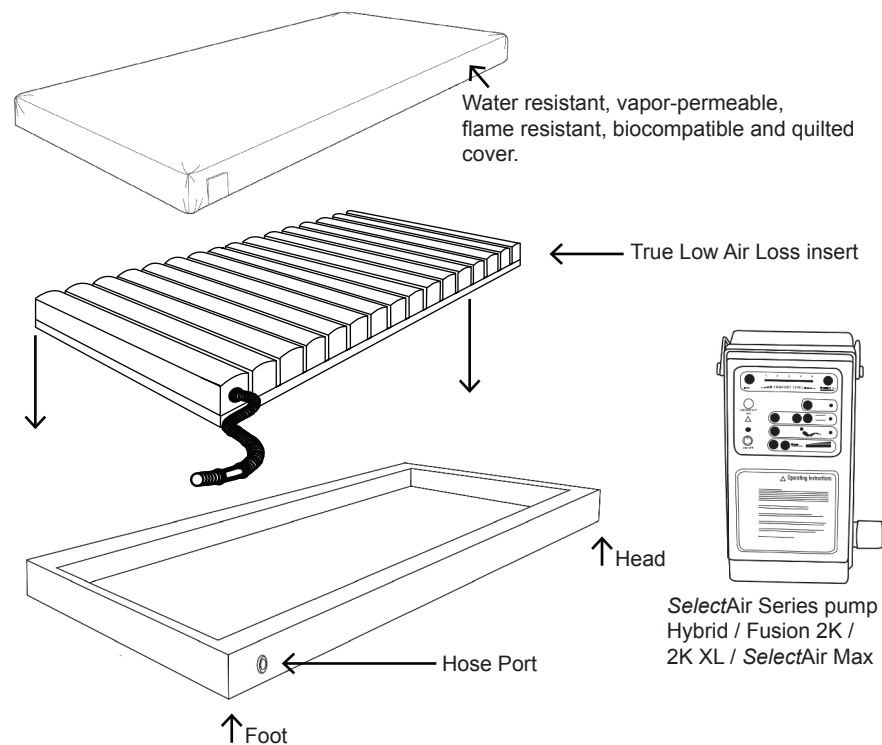### Safety Precautions:

The use of bed frame side rails in conjunction with any FUSION 2K mattress system should be determined based upon facility protocol, applicable laws and regulations, and individual user safety considerations.

Bed frame side rail usage is not a requirement for the functional operation of any FUSION 2K product, but individual safety requirements and/or facility protocol may dictate that side rails be utilized.

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